INSTRUCTIONS FOR USE

INTENDED USE

CVA (Calibration Verification Assessment) and CVA for CELL-DYN are assayed linearity control kits which can be used to assess calibration and to verify patient reportable ranges of three and five-part hematology instruments, as well as, automated and semi-automated non-differential hematology analyzers.

SUMMARY AND PRINCIPLES

CAP requirements¹ and CLIA regulations² both mandate that laboratories substantiate their test methods throughout the reportable range for patient test results upon installation of an analyzer. Other circumstances which may call for calibration verification include major preventive maintenance, unusual trends in control performance, or whenever recommended by the instrument manufacturer. CVA contains WBC, RBC/Hgb and Platelet concentrations which span typical patient reportable ranges; allowing the user to comply with these guidelines. CVA is vialled as separate components enabling calibration assessment without matrix interference.

Assay values are derived from replicate testing on instruments operated and maintained according to manufacturer's instructions.

REAGENTS

CVA and CVA for CELL-DYN contain stabilized human red and white blood cells and simulated or human platelets vialled as separate components in buffered aqueous media.

PRECAUTIONS

- For In Vitro Diagnostic Use.
- All human source material used to manufacture this product was non-reactive for antigens to Hepatitis B (HBsAg), negative by tests for antibodies to HIV (HIV-1/HIV-2) and Hepatitis C (HCV), non-reactive for HIV-1 RNA and HCV RNA by licensed NAT, and non-reactive to Serological Test for Syphilis (STS) using techniques specified by the U.S. Food and Drug Administration. Because no known test method can assure complete absence of human pathogens, this product should be handled with appropriate precautions.
- This product should not be disposed in general waste, but should be disposed with infectious medical waste. Disposal by incineration is recommended.
- CVA is not to be used for instrument calibration.

STORAGE AND STABILITY

CVA/CVA for CELL-DYN is stable through the expiration date when stored at 2° to 10°C. After opening, the product is stable throughout the open-vial dating, as indicated on the assay sheet, when stored at 2° to 10°C.

INDICATION OF PRODUCT DETERIORATION

Inability to obtain assay values may indicate product deterioration. To determine the source of potential problems:

- Check the expiration date and discard outdated product. Discoloration of the product may be caused by overheating or freezing during shipping or storage. Gross hemolysis (darkly colored supernatant) may indicate product deterioration. However, moderately colored supernatant is normal and should not be mistaken for product deterioration.
- Review the operating procedures of your instrument.
- If problems persist, contact Technical Services, Streck, 800-843-0912 or at www.streck.com.

INSTRUCTIONS FOR USE

Preparation

- Perform routine maintenance, quality control procedures, calibration and precision checks according to your laboratory protocol. If acceptable, proceed with running CVA or CVA for CELL-DYN.
- Remove vials of control from refrigerator and warm to room temperature (18° to 30°C) for 15 minutes before use.
- Ensure that sufficient, unexpired reagents are available and that no patient samples will be run on the instrument while running CVA or CVA for CELL-DYN.

Mixing and Handling

- Do not mix mechanically or vortex, unless otherwise noted.
 - Hold vial horizontally between the palms of the hands and roll the vial back and forth for 30 seconds.
 - Mix vigorously by holding the vial at one end and flicking the wrist (similar to shaking down a thermometer) for 60 seconds. Repeat as many times as necessary until all cells are suspended.
 - Continue to invert tube vertically end over end, 15-20 times, after the button of cells at the bottom of the vial is no longer visible and making sure that no cell clump is visible in the vial.
 - Invert the vials 8-10 times between each sampling from the vial.
 - Vials stored for an extended period of time and vials with extremely high concentration of cells may require extra mixing (especially for vials of Platelets and WBCs).

Important Notes for CVA

- Siemens Healthcare Diagnostics ADVIA® 120/2120 Hematology System – Refer to WBCB for white blood cell count.
- Beckman Coulter® instruments – Disable the differential and blood detector (if applicable) before running CVA. (CVA may be run in primary or secondary mode.)

Important Notes for CVA for CELL-DYN

- Special mixing note for CVA for CELL-DYN platelets: The CVA for CELL-DYN platelet vials require a 30-second thorough mixing using a vortex mixer prior to sampling. If a vortex mixer is not available, vigorously mix the vials by hand for 30 seconds prior to sampling until all cells have been suspended.
- For the Emerald instrument, run CVA for CELL-DYN in the instrument's Linearity Mode.

Analysis Protocol for CVA and CVA for CELL-DYN

- Perform a background count and, if acceptable, proceed.
- Refer to the assay sheet for dilution preparation or special instructions for the following instruments: Abbott CELL-DYN® 3500, 3700, 3200, 1600, Ruby and Emerald, Beckman Coulter® STKR/STKS, MAXM, A^CT™ 5diff, LH 500 and Mindray BC-3200.
- Analyze all levels from one component set in order of lowest concentration to highest concentration. Run samples in Patient Mode, unless otherwise noted.
- Run each vial four times, mixing before each aspiration. Record data on data sheet and calculate the average (mean) values.
- Repeat steps 1-3 for remaining components.
- Perform a background count before running each component set.
- After sampling, return to refrigeration for maximum open-vial stability. If run in the open mode, wipe the threads of both vial and cap before replacing cap and returning to refrigeration.
- Instruments with assay-specific values should verify that the mean values recovered are within the expected ranges.
 - If the mean values are outside of your laboratory's established acceptance limits, rerun CVA.

- If the mean values remain outside of your established limits, contact your instrument service representative. Do not adjust your instrument.
- Record values for all runs on the worksheet and submit data to Streck STATS® Department. **Data is processed weekly and due at Streck by 5:00 p.m. CST on Thursday to be processed on Friday.**

Dilutions

- Dilutions - Assay: Ensure that the appropriate sample dilutions have been performed as stated on the assay sheet.
- OPTIONAL - Dilutions - All instruments: Customers desiring a linearity graph that extends lower than the assay values can perform a serial dilution according to the dilution ratios listed on the enclosed worksheet.

LIMITATIONS

A manual differential analysis of white blood cells cannot be accomplished with this product. The white blood cells simulate white blood cells in size, not morphology.

EXPECTED RESULTS

Values recovered should fall within the suggested tolerance limits on the accompanying assay. Due to lab-to-lab variation, some instruments may not recover values within the stated assay ranges for levels that are exceptionally concentrated or dilute. Instrument calibration, reagents and operator technique may yield differences in results obtained. Also, each laboratory must evaluate the data and use it as a guide in determining its level of instrument or testing accuracy, as well as evaluating acceptable reportable ranges. Performance goals for linearity should also be established by a laboratory based on clinical needs and what is technically achievable with instrumentation. Limits should not exceed the instrument manufacturer's linearity specification stated in the instrument manual.

- If the mean values are outside of your laboratory's established acceptance limits, rerun CVA.
- If the mean values remain outside your established limits, contact your instrument service representative. **DO NOT ADJUST YOUR INSTRUMENT.**

The following reference methods are used to whole blood calibrate instruments prior to system specific value assignment.

WBC - A series of 1: 500 dilutions are made using class A glassware. The lytic reagent is placed in the initial dilution flask before diluting to volume. The diluting agent is an isotonic solution for Beckman Coulter instruments. The samples are counted on a Beckman Coulter Counter Z series instrument.

RBC - A series of 1: 50,000 dilutions are made using class A glassware. The diluting agent is an isotonic solution for Beckman Coulter series instruments. The samples are counted on a Beckman Coulter Counter Z series instrument.

Hgb - Hemoglobin concentration is determined by converting hemoglobin to hemoglobinocyanide (HiCN) and measuring absorbance at 540nm according to ⁴NCCLS H15-A3 and ICSH recommendations. Hemoglobin concentration is calculated using millimolar absorption coefficient of 11.0.

Plt - A series of 1:125 macrodilutions are prepared using class A glassware in 1% Ammonium Oxalate. Charged hemacytometers are allowed to stand 20-30 minutes. Cells are counted using phase-contrast microscopy technique.

REFERENCES

- College of American Pathologists, 1994, Commission on Laboratory Accreditation Inspection Checklist HEM.25760.
- Clinical Laboratory Improvement Amendments of 1988, final rule: 42 CFR 493.1253.
- Beckman Coulter Counter Model ZBi with attached flat pack, Beckman Coulter Counter Model MHR MCV/Hct/RBC Computer.
- Clinical Laboratory Standards Institute. Reference and Selected Procedures for the Quantitative Determination of Hemoglobin in Blood- Second Edition. Approved Standard.

QUALITY CONTROL

Streck offers STATS®, an interlaboratory quality control program, and STATS-Link®, which provides internet access to STATS reports, to all qualifying customers at no charge. If you are interested in more information or would like to participate, contact the STATS Department at 800-898-9563, or by fax at 402-333-7874. Additional information can be found online at www.streck.com.

ORDERING INFORMATION

Please call our Customer Service Department toll free 800-228-6090 for assistance. Additional information can be found online at www.streck.com.

GLOSSARY OF HARMONIZED SYMBOLS

 Authorized Representative in the European Community	 Batch Code	 Biological Risk	 Catalog Number	 Use By
 In Vitro Diagnostic Medical Device	 Manufacturer	 Consult Instructions For Use	 Temperature Limitation	

Glossary of symbols may contain symbols not used in the labeling of this product.

The brand and product names of the instruments are trademarks of their respective holders.

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QUELLENANGABEN

- 1College of American Pathologists, 1994, Commission on Laboratory Accreditation Inspection Checklist HEM.25760.
- 2Clinical Laboratory Improvement Amendments of 1988, final rule: 42 CFR 493.1253.
- 3Beckman Coulter Counter Model ZBi with attached flat pack, Beckman Coulter Counter Model MHR MCV/Hct/RBC Computer.
- 4Clinical Laboratory Standards Institute. Reference and Selected Procedures for the Quantitative Determination of Hemoglobin in Blood- Second Edition. Genehmigte Normen.

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BIBLIOGRAFIA

- 1 College of American Pathologists, 1994, Commission on Laboratory Accreditation Inspection Checklist HEM.25760.
- 2 Clinical Laboratory Improvement Amendments of 1988, final rule: 42 CFR 493.1253.
- 3 Beckman Coulter Counter Model ZBi with attached flat pack, Beckman Coulter Counter Model MHR MCV/Hct/RBC Computer.
- 4 Clinical Laboratory Standards Institute. Reference and Selected Procedures for the Quantitative Determination of Hemoglobin in Blood- Second Edition. Approved Standard.

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REFERANSER

- ¹College of American Pathologists, 1994, Commission on Laboratory Accreditation Inspection Checklist HEM.25760.
- ²Clinical Laboratory Improvement Amendments of 1988, final rule: 42 CFR 493.1253.
- ³Beckman Coulter telleinstrument modell ZBI med festet "flat pack", Beckman Coulter telleinstrument modell MHR MCV/Hct/RBC datamaskin.
- ⁴Clinical Laboratory Standards Institute. Reference and Selected Procedures for the Quantitative Determination of Hemoglobin in Blood - Second Edition. Approved Standard.

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- 1 College of American Pathologists, 1994. Commission on Laboratory Accreditation Inspection Checklist HEM.25760.
- 2 Clinical Laboratory Improvement Amendments of 1988, final rule: 42 CFR 493.1253.
- 3 Beckman Coulter Counter Model ZBI with attached flat pack, Beckman Coulter Counter Model MHR MCV/Hct/RBC Computer.
- 4 Clinical Laboratory Standards Institute. Reference and Selected Procedures for the Quantitative Determination of Hemoglobin in Blood. Segunda edición. Norma aprobada.

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REFERENSER

- ¹ College of American Pathologists, 1994, Commission on Laboratory Accreditation Inspection Checklist HEM.25760.
- ² Clinical Laboratory Improvement Amendments of 1988, final rule: 42 CFR 493.1253.
- ³ Beckman Coulter Counter Model ZBI with attached flat pack, Beckman Coulter Counter Model MHR MCV/Hct/RBC Computer.
- ⁴ Clinical Laboratory Standards Institute. Reference and Selected Procedures for the Quantitative Determination of Hemoglobin in Blood- Second Edition. Godkänd standard.

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