# Hematology Calibrators

Hematologické kalibrátory / Les calibrateurs Hématologie Streck / Die Streck Hämatologie-Kalibratoren / I Calibratori ematologici Streck / Streck Hematologikalibratoren / Los Calibradores Hematológicos Streck / Streck Hematologikalibratoren

Cal-Chex, Cal-Chex A Plus, Cal-Chex•H, Cal-Chex X

# Streck

## INSTRUCTIONS FOR USE

### INTENDED USE

Streck Hematology Calibrators are manufactured for calibrating multi-parameter hematology analyzers.

### SUMMARY AND PRINCIPLES

Multi-parameter hematology analyzers require regular calibration in order to produce accurate results on patient samples. Calibration can be accomplished by transferring information to the analyzer through fresh blood samples which have been assayed by reference methods. A more direct and convenient approach is to use a calibrator material with System Specific Values (SSV) assigned.

Streck Calibrators are stable suspensions of red blood cells, white blood cells and platelets. Assigned values are derived from replicate analyses on whole-blood calibrated hematology analyzers. Users analyze the calibrator on their instruments and compute calibration factors by comparing recovered values and assigned values. These factors provide the basis for making adjustments to the instrument.

### REAGENTS

This whole blood reagent may contain any or all of the following: stabilized human or animal red blood cells, human, animal or simulated white blood cells and a platelet component in a preservative medium.

### PRECAUTIONS

1. For In Vitro Diagnostic Use.
2. All human source material used to manufacture this product was non-reactive for antigens to Hepatitis B (HBsAg), negative by tests for antibodies to HIV (HIV-1/HIV-2) and Hepatitis C (HCV), non-reactive for HIV-1 RNA and HCV RNA by licensed NAT, and non-reactive to Serological Test for Syphilis (STS) using techniques specified by the U.S. Food and Drug Administration. Because no known test method can assure complete absence of human pathogens, this product should be handled with appropriate precautions.
3. This product should not be disposed of in general waste, but should be disposed of with infectious medical waste. Disposal by incineration is recommended.
4. This product is intended for use as supplied. Adulteration by dilution or addition of any materials to the product as supplied invalidates any diagnostic use of the product.

### STORAGE AND STABILITY

Streck Calibrators are stable through the expiration date when stored at 2° to 10°C. After opening, the product is stable throughout the open-vial dating, as indicated on the assay sheet, when stored at 2° to 10°C.

### INDICATION OF PRODUCT DETERIORATION

Gross hemolysis (darkly colored supernatant) may be indicative of product deterioration or damage. If this is apparent, contact Streck Technical Services at 800-843-0912 or online at [technicalservices@streck.com](mailto:technicalservices@streck.com).

### INSTRUCTIONS FOR USE

1. Perform instrument start-up and routine cleaning procedures as defined in instrument manufacturer's operator's manual (i.e., blood sampling valve, counting apertures).
2. Remove vials of calibrator from refrigeration and warm to room temperature (18° to 30° C) for 15 minutes before use.
3. To Mix: (Do not mix mechanically.)
  - a. Hold vial horizontally between the palms of the hands and roll the vial back and forth for 20 to 30 seconds.
  - b. Mix by rapid inversion to ensure the cells are suspended.
  - c. Vials stored for an extended period of time may require extra mixing.
  - d. Gently invert the vials 8 to 10 times immediately before sampling.
4. Prime the instrument by aspirating two normal blood samples. Disregard results.
5. Verify instrument precision. Refer to operator's manual for instructions and specifications.
6. Analyze the calibrator 11 times, disregard results of first run. (For auto-calibration, refer to procedure in operator's manual.) Record the values recovered on remaining 10 runs. (Analyzing the calibrator 11 times is recommended. However, some instruments may require fewer analyses.)
7. Calculate the mean value for each parameter.
8. a. Compare the results of your calculations to the values listed for your instrument type.
  - b. If the difference between your recovered mean values and the system specific values are less than the listed tolerance limits, the instrument does not require calibration.
  - c. If the difference is greater, calibrate using the system specific values.
9. Calibration of the specific parameter(s) should be done in accordance with the procedure in your instrument manual. Calibration may not be required for all parameters.
10. Verify calibration by analyzing the calibrator three times and repeat steps 7 and 8.
11. For further assistance, contact Streck Technical Services at 800-843-0912 or online at [www.streck.com](http://www.streck.com).
12. After sampling, return to refrigeration for maximum open-vial stability. If run in the open mode, wipe the threads of both the vial and cap before replacing cap and returning to refrigeration.

### EXPECTED RESULTS

Assay values are derived from replicate testing on instruments operated and maintained according to manufacturer's instructions.

The following reference methods are used to whole blood calibrate instruments prior to system specific value assignment as indicated in CLSI H26-A2.<sup>1</sup>

- WBC** - A series of 1:500 dilutions are made using class A glassware. The lytic agent is placed in the initial dilution flask before diluting to volume. The diluting agent is an isotonic non-interfering solution. The samples are counted on a single aperture volume displacing particle counter.
- RBC** - A series of 1:50,000 dilutions are made using class A glassware. The diluting agent is an isotonic non-interfering solution. The samples are counted on a single aperture volume displacing particle counter.
- Hb** - Hemoglobin concentration is determined by converting hemoglobin to hemoglobincyanide (HiCN) and measuring absorbance at 540nm according to CLSI H15-A3<sup>2</sup> and ICSH recommendations. Hemoglobin concentration is calculated using millimolar absorption of 11.0.
- Hct** - Microhematocrit values are done in replicate on each sample, with capillary tubes filled and centrifuged according to the CLSI H07-A3<sup>3</sup> document. K2EDTA is used as the anticoagulant for the collection of fresh specimens. The packed cell volume (hematocrit) is read directly using a precision metric scale. No correction is made for trapped plasma.
- Plt** - Platelet counting by the RBC/platelet ratio method (ICSH) is utilized as the reference method.


### REFERENCES

- <sup>1</sup>Clinical Laboratory Standards Institute. Validation, Verification and Quality Assurance of Automated Hematology Analyzers-Second Edition. Approved Standard.
- <sup>2</sup>Clinical Laboratory Standards Institute. Reference and Selected Procedures for the Quantitative Determination of Hemoglobin in Blood-Third Edition. Approved Standard.
- <sup>3</sup>Clinical Laboratory Standards Institute. Procedure for Determining Packed Cell Volume by the Microhematocrit Method-Third Edition. Approved Standard.

### ORDERING INFORMATION

Please call our Customer Service Department toll free 800-228-6090 for assistance. Additional information can be found online at [www.streck.com](http://www.streck.com).

## GLOSSARY OF HARMONIZED SYMBOLS

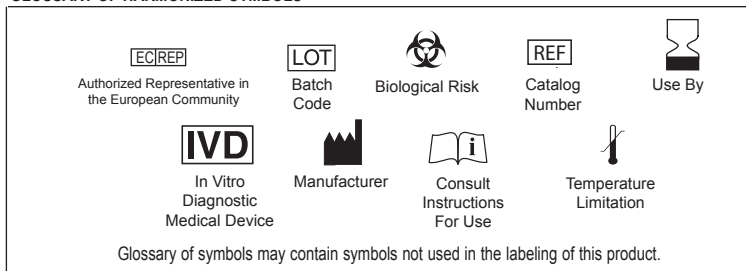The brand and product names of the instruments are trademarks of their respective holders.

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