INSTRUCTIONS FOR USE

The Temp-Chex Digital thermometer assembly is designed as a convenient device for monitoring the temperature when storing a variety of products including clinical laboratory reagents, controls, and patient specimens.

PRINCIPLE

Temp-Chex Digital is comprised of a digital thermometer base unit and accompanying stainless steel temperature probe. This thermometer is manufactured to meet NIST/DKD standards of accuracy and is certified traceable to NIST/DKD standards. Temp-Chex Digital is accurate to $\pm 1^\circ\text{C}$ between -50 and 120°C , $\pm 2^\circ\text{C}$ between 120°C and 150°C , and $\pm 4^\circ\text{C}$ between 150°C and 200°C . Temp-Chex Digital is not a NIST or DKD thermometer.

COMPONENTS

The stainless steel temperature probe comes within a pre-assembled glass vial/probe unit containing glass beads as filler material. This glass vial/probe unit connects to the thermometer base unit through the use of a metal cord. The base unit can be attached magnetically, with adhesive, or by keyhole, to any compatible exterior surface.

CALIBRATION STATEMENT

In our opinion once calibrated and used with proper care, your digital thermometer should maintain its accuracy. Due to use in diverse applications and handling there is no precise way to predict how long calibration will be maintained. Accuracy can be affected by shock, aging, temperature and contamination. It is the user's responsibility to maintain traceability through periodic recalibrations or verifications. Contact Streck Sales at 1-800-843-0912.

INSTRUCTIONS FOR USE

- Unpack thermometer base unit, pull down the battery cover, and remove the insulation strip in the battery compartment. Insert the enclosed AAA battery into unit. Unpack glass vial/probe unit and connect to base unit through use of the connector cord. Turn on the power by pressing the [ON / OFF] button. The unit is now ready to be placed into service.
- Place glass vial/probe unit inside of the temperature-controlled area. Mount the base unit outside of this area. The base unit should not be exposed to extreme temperatures. DO NOT immerse glass vial/probe unit in liquid (e.g., water baths).
- Prior to attachment of Temp-Chex Digital base unit, determine if the wall is ferrous metal and will allow use of the magnetic section. If so, make sure the surface is clean and free of any moisture or frost. Place the enclosed magnet on the surface and then attach the Temp-Chex Digital base unit. Verify that it will hold properly before placing into service.

If the wall is not ferrous metal or if the surface does not provide proper magnetic attraction, the adhesive surface of the enclosed magnets can be used. Once again, the surface should be clean and dry. First check that the two magnetic parts adhere to each other (if incorrectly aligned, the magnets will repel each other). Maintain proper orientation of the magnetic strip when attaching it to the freezer, incubator, or refrigerator wall. Remove paper backing from the second magnet, attach the adhesive side to the wall and allow a few minutes to establish the appropriate bonding prior to attaching the magnet already affixed to the Temp-Chex Digital base unit. It is safest to attach the base unit to walls in such a position that a shelf can support it. However, provided the appropriate attachment procedure has been followed, the base unit should remain stationary.

MEASUREMENTS

Turn base unit on. Record the reading once the temperature has become stable.

FUNCTIONS

$^\circ\text{C}$ / $^\circ\text{F}$ EXCHANGE:

Press the [$^\circ\text{C}$ / $^\circ\text{F}$] button once.

MAXIMUM / MINIMUM MEMORY:

Press [MAX / MIN] button once to display the maximum reading and a MAX symbol will be flashing. Press [MAX / MIN] once more to show the minimum reading and a MIN symbol will be flashing. Press [MAX / MIN] button again to return to normal display.

CLEAR MAXIMUM / MINIMUM TEMPERATURE MEMORY:

Turn the Temp-Chex Digital unit off to clear the memory.

DATA HOLD:

Press the [HOLD] button once to retain the display reading and the unit scale indication symbol will be flashing. Press [HOLD] button again to return to normal display.

POWER ON / OFF:

Press [ON / OFF] button once.

BATTERY

Replace the AAA battery if:

- An "A" appears in the corner of the display.
- The display reads "DIM"
- There is no display or a dim display (the battery may be discharged)
- Readings are inconsistent

The battery life for this unit is typically one year, however, life expectancy depends on usage.

TEMPERATURE DISPLAYS:

- If the thermometer is placed outside of the range of designed use, the display may read the following: "LLL" when used below -50°C , "HHH" when used above 200°C
- If using the thermometer within the range of designed use and the display reads "LLL", or "HHH" make sure the probe is inserted correctly, turn the unit off and then back on.

NOTE:

- When the MAX / MIN reading is being displayed or the HOLD function is activated, the thermometer cannot be powered off until all of the functions are released.
- Do not remove probe from unit. If the probe is disconnected from the glass vial/probe unit during use, reconnect the probe and then power the unit off and on to receive correct readings.
- Avoid transferring probe unit directly from one extreme temperature environment to another (e.g., freezer to oven); instead, allow unit to slowly adjust to room temperature first.
- If probe insertion within the glass vial/probe unit becomes offset relative to its original position, adjust insertion such that approximately $1\frac{3}{16}$ inch (30 mm) of the probe extends above the top surface of the vial.

SPECIFICATIONS

Measurement Range: -50 to 200°C / -58 to 392°F

Accuracy:

$\pm 1.0^\circ\text{C}$ between -50°C and 120°C .

$\pm 2.0^\circ\text{C}$ between 120°C and 150°C .

$\pm 4.0^\circ\text{C}$ between 150°C and 200°C .

Display resolution: 0.1°C between -19.9°C and 199°C , otherwise 1°C

Sampling rate: 1.0 second

Battery: 1.5V AAA

Display size: 36mm (W) x 16mm (H)

Probe size: 65mm (L)

Probe cord length: 1524mm

Product size: 65mm (W) x 97mm (H) x 20mm (D)

Product weight: $\approx 130\text{g}$

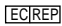
ORDERING INFORMATION

Please call our Customer Service department toll free 800-228-6090 for assistance. Additional information can be found online at www.streck.com.

INTERNET CERTIFICATE OF CALIBRATION DIRECTIONS

A translated, generic Certificate of Calibration can be obtained via the Internet at www.streck.com using the Temp-Chex Digital product page, or by calling the following phone number, 1-800-843-0912, or by calling your local supplier.

GLOSSARY OF HARMONIZED SYMBOLS

 Authorized Representative in the European Community	 Batch Code	 Biological Risk	 Catalog Number	 Use By
 In Vitro Diagnostic Medical Device	 Manufacturer	 Consult Instructions For Use	 Temperature Limitation	 WEEE

Glossary of symbols may contain symbols not used in the labeling of this product.

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