


Cyto-Chex[®] BCT

MIXING PATIENT SAMPLE AFTER COLLECTION

- Collect specimen by venipuncture according to CLSI document H3-A6
- Follow recommendations for order of draw outlined in CLSI document H3-A6
- Fill tube completely
- Remove tube from adapter and immediately mix by **gentle inversion 8 to 10 times**
- **Inadequate or delayed mixing may result in inaccurate test results**

INVERT IMMEDIATELY AFTER BLOOD IS DRAWN

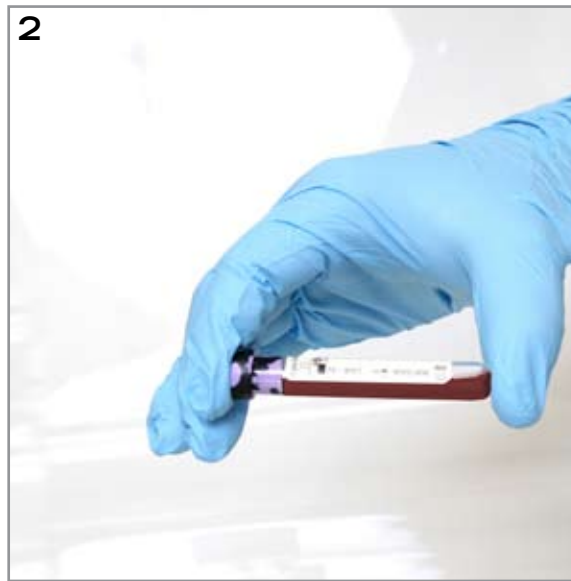

REFERENCE

1. CLSI document H3-A6, Procedures for the Collection of Diagnostic Blood Specimens by Venipuncture