


## ESR-Auto Plus Operator's Manual Addendum

July 2010, 880020

The following is a supplemental addendum to the ESR-Auto Plus Operator's Manual that explains new features included on ESR-Auto Plus instruments containing software version V0002(100618). It is meant to be used in conjunction with ESR-Auto Plus Operator's Manual (p/n 320578). Please refer to this addendum for information concerning the Audible Alarm function and the QC Menu functions to Edit Control Ranges and Clear the QC Log File.


### Edit Control Ranges

Only control levels previously registered in the Register Controls Menu are available in the Edit Control Ranges Menu. For example, if only Level 1 control values have been registered, then the Edit Control Ranges Menu will allow edits to the minimum and maximum values for Level 1, but will not display any Level 2 information. If no controls have been registered, the instrument will display "No Control Data Registered" and no changes will be allowed.

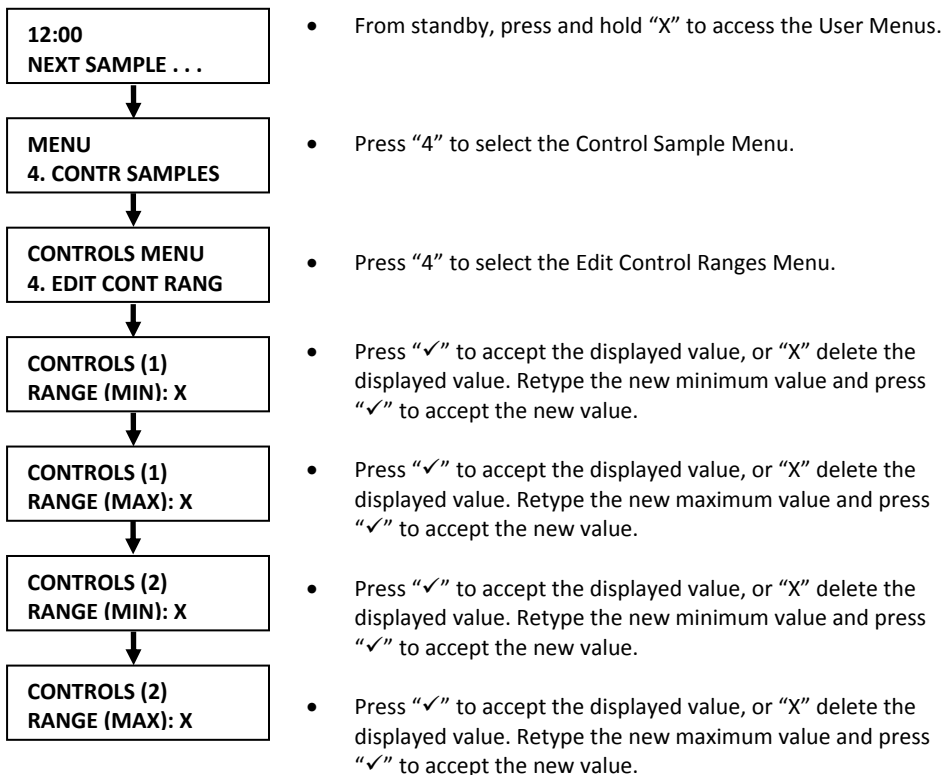

### Clear QC Log File

All daily QC data contained in the QC log file may be cleared in its entirety using the Clear QC Log function in the Show Controls Menu. Control lot information, such as lot number and min/max ranges, will not be affected by clearing the QC log file.

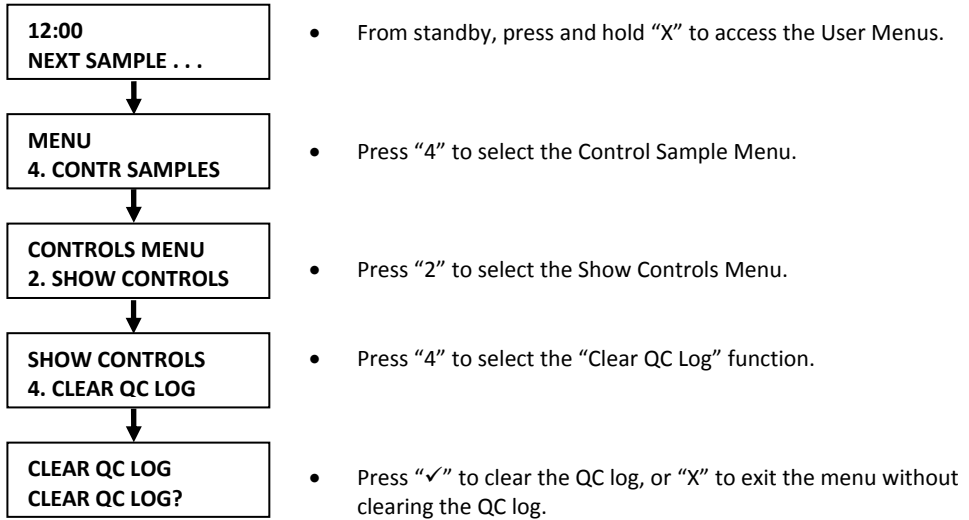

### Audible Alarm

The ESR-Auto Plus contains an optional audible alarm to signal the conclusion of each test result. The audible alarm function can be activated or deactivated in the User Configuration Menu as outlined below.

