

ESR-Auto Plus® Repackaging Instructions

Feb 2007, 880007

The ESR-Auto Plus contains delicate electrical components and needs to be properly packed in the approved foam inserts and carton before shipping. Please contact Streck to obtain the correct packaging materials if needed.

Please follow these instructions carefully:

1. Unplug the instrument. Remove the paper, test rack, cables, and other accessories.
2. Clean the instrument and test rack by wiping the outer surface with an ethanol or isopropanol solution.
3. Place the bottom foam piece in the shipping carton. (See Figure 1)
4. Insert the test rack into the ESR-Auto Plus and place the instrument inside the antistatic bag supplied with the approved packaging.
5. Place the instrument and mixer in the designated slot in the bottom foam piece. (See Figure 2)
6. Place the top foam piece in the shipping carton with the flat side down. (See Figure 3)
7. Place any accessories inside the top foam piece. (See Figure 4)
8. Fill any extra space with packing material before closing the box.
9. Always ship the instruments and components as "FRAGILE"!

Please contact Streck at 1.800.843.0912 x7510 with any questions or concerns.



Figure 1



Figure 2



Figure 3



Figure 4