








ESR-Auto Plus[®] Testrack Registration Procedure

Jan 2007, 880005

1. Unplug the instrument. Wait at least 5 seconds and plug in the instrument.
2. Instrument reads: *"Initiating Motor..."*
3. Instrument reads: *"Next Sample..."*
Press the Red  button.
Enter "6"
4. Instrument reads: *"User Config"*
Press the red  button.
5. Instrument reads: *"Service Menu"*
Press the green  button.
6. Instrument reads: *"Give Code"*
Enter "48"
Press the green  button.
Press the red  button until the instrument reads *"Reg Test Rack?"*
7. Instrument reads: *"Reg Test Rack?"*
Press the green  button.
8. Instrument reads: *"Sure?"*
Press the green  button.
The instrument will scan twice and print results.
The printout should read "TEST RACK REGISTRATION OK".
9. Instrument reads: *"Set Thermometer?"*
Press the red  button.
10. Instrument reads: *"Test Mode On?"*
Press the red  button.
11. Instrument reads: *"Reading Test?"*
Press the red  button.
12. Instrument reads: *"Power Test?"*
Press the red  button.
13. Instrument reads: *"Next Sample..."*

The instrument is ready to run samples.