

Instructions for assay zip file download

Windows operating systems

1. Insert a formatted USB flash drive or a formatted 3½-inch floppy disk into the computer (see detail at right).
 2. Go to the product page at streck.com and download the assay zip file for the lot number you are using (see detail at right).
 3. When prompted, click “Open” to download and open the file.
 4. Once the file is opened, click “File” → “Extract All” or click “Extract All...”.
- *Note: Windows 7 Users - Be sure the box reading “Show extracted files when complete” is UNCHECKED.
5. Follow the “Extraction Wizard” (if applicable). Click on the “Browse” button to select an extract destination. Select the flash drive or floppy disk that was inserted in step 1. Click “Extract.”
 6. After the files have been extracted, safely remove the flash drive or floppy disk from the computer and insert into the appropriate instrument.
 7. Refer to instrument operator’s manual for assay file set up.


Mac operating systems

1. Insert a formatted USB flash drive or a formatted 3½-inch floppy disk into the computer.
2. Go to the product page at streck.com and download the assay zip file for the lot number you are using. The file should download to your default download folder (“Downloads” on most systems).
3. Once the file is downloaded, locate the file and double click it. The zip file will be extracted to a folder with the same title as the zip file. Open this folder.
4. Click “Edit” on the top menu bar and click “Select All.” Click “Edit” again and click “Copy.”
5. On the computer, locate the folder for the flash drive or floppy disk. Open the folder and click “Edit” on the top menu bar and click “Paste.”
6. After the files have been copied, safely remove the flash drive or floppy disk from the computer and insert into the appropriate instrument.
7. Refer to instrument operator’s manual for assay file set up.

Instructions to format a disk/USB flash drive

- Click on “Start,” then “(My) Computer.”
- Right click on the media to format (3½-inch floppy or removable disk).
- Click on “Format.”
- Under Format Options, select “Quick Format,” then “Start.”

Quick access to the product page
Click on the search icon.

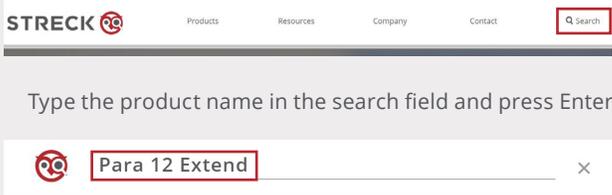

Type the product name in the search field and press Enter.

Click on product name in the search results.



On the product page, scroll down and click Resources in the menu and then “Assay Zip File Downloads.”



Resources

<p>Instructions (IFU)</p> <ul style="list-style-type: none"> Para 12 Extend IFU FDA Required Blood Tests Para 12 Extend Mixing Instructions Video Glossary of Symbols 	<p>Assays</p> <ul style="list-style-type: none"> Para 12 Extend Para 12 Extend Para 12 Extend
<p>SDS</p> <ul style="list-style-type: none"> Para 12 Extend SDS Exemption Letter 	<p>STATS</p> <ul style="list-style-type: none"> STATS Data Inp STATS Data Inp STATS Due Dat

Assay Zip File Downloads (Abbott CELL-DYN 1700 & 1800 only)

- Instructions for Assay Zip File Download

Select lot-specific zipped folder to download.